

## What is the 2ml Single Kit?

The 2ml single kit has been developed to support Commissioners and Drug services to provide a product that not only promotes best practice and improves value for money, but also provides the best possible equipment available to ensure clients are injecting and disposing of equipment in the safest way possible. This kit enables clients to take only what they need, therefore reducing unused drug litter created from clients having to take larger packs with more equipment than needed and disposing of over requirement in the community. More importantly it has a single use sharps container that enables safe disposal of the used injecting equipment. The single use sharps container is so small it enables safe carrying and disposal at point of use.

The single use kit directly promotes non-sharing of equipment to ensure minimum exposure to Blood Borne viruses. It achieves this by providing all equipment needed in individual packaging to carry out one injection. Once used it is disposed of in the single use sharps container immediately after an injection and locked away safe unable to be re-used. All the equipment provided is guaranteed sterile ready for use until opened.

This kit contains the new SAFELOC<sup>™</sup>two, high quality single use sharps container for the safe and secure disposal of a single used needle & syringe. This single use container has been produced specifically for the Harm Reduction Market to help eliminate used drug equipment disposed of in the community, and the accidental re-use of used equipment. This results in reduced risks of BBV transmissions.


### Contents:

The contents of this kit are unused and sterile until opened. Do not share this equipment with anyone or re-use the equipment again. Please ensure you dispose of all litter in the bag provided with the kit and that the used needle & syringe is made safe and secure in the SAFELOC<sup>™</sup>two, container provided.

- 1 x 2ml coloured ultra low dead space luer slip syringe with scratch off identification symbols
- 1 x hypodermic needle
- 1 x sterile spoon with filter
- 1 x sachet of acid
- 1 x alcohol swab
- 1 x SAFELOC<sup>™</sup> two, single sharps container
- 1 x 2ml Ampoule of water (optional)\*

## Logistics

<b>Pcs per carton:</b>	260
<b>Carton Dimensions (mm):</b>	375(w) x 365(d) x 310(h)
<b>Carton Weight:</b>	6.20kg

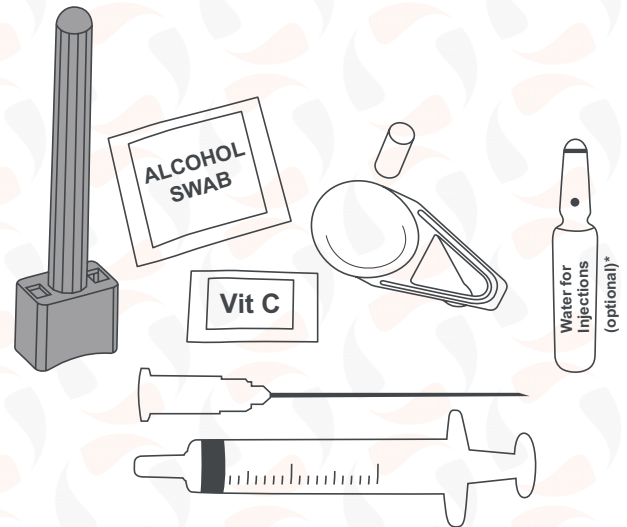

## Benefits of the 2ml Single Kit


- Encourages safer injecting practice
- Reduces the spread of BBVs
- Promotes non sharing through single use items
- Provides individually wrapped clean sterile equipment
- Increases use of sharps containers
- Delivers improved value for money
- Reduces used drug litter
- Reduces unused equipment waste
- Complies with best practice & NICE PH52 Recommendations
- **Eliminates accidental sharing and re-using of equipment**
- **Promotes point of use disposal**

## How to use the 2ml Single Kit


- 1 Open the bag by tearing along the serrated line, and remove all contents and prepare for injecting.



- 2 On completion of the injecting episode replace all used paraphernalia back into the plastic bag and dispose of safely.

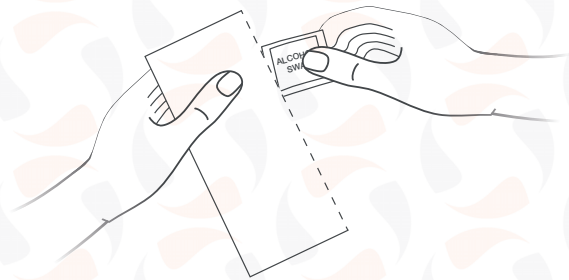

- 3 Using the SAFELOC<sup>™</sup> two, single sharps container provided, place your used needle & syringe needle first into the container and push down with your thumb or finger until you feel and hear an audible click. Once you feel and hear the needle & syringe click in place, the used needle & syringe is now secure in the container. The container is now safe to carry in a pocket or bag until such time it can be disposed safely at a pharmacy or drug service.

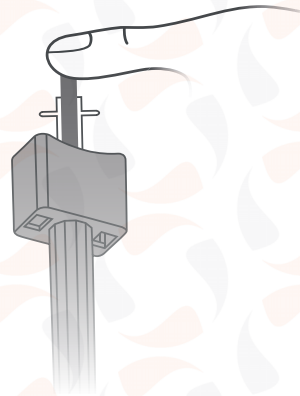

### \*Note

If you would require water to be part of this kit also, please contact Orion for a price of a kit with water, details are below.

Additionally Orion has the packing capability to pack bespoke single kits to your own requirements. We would be pleased to discuss and help with your requirements.